



BENEFITS

- Cleaning and disinfection by suction of saliva pumps
- Bactericidal, yeasticidal and virucidal



Formule douce



APPLICATION

Cleaning and disinfection by suction of surgical suction circuits and saliva pumps. Controlled foaming action. Medical device. CLASS IIA.

INSTRUCTION FOR USE

BACTINYL® ASPIRATION CHIRURGICALE is designed for the cleaning and disinfection of suction systems (surgical suction, saliva pump ...) in hospitals, clinics, dental surgeries ...

Dilute BACTINYL® ASPIRATION CHIRURGICALE to 2% in cold or warm water. Aspirate this solution with the flexible tips of the suction systems. Leave to act for 15 minutes. Rinse.

CHARACTERISTICS

Appearance :	liquid
Colour :	colourless
Perfume :	odourless
Density :	= 1
pH :	10.20
Expiration :	2 years
Storage :	4°C - 35°C

MICROBIOLOGICAL ACTIVITY

Bactericidal : EN 13727 (dirtiness conditions – 0.5% – 5 min – 20°C), EN 14561 (dirtiness conditions – 1% – 5 min – 20°C),

Yeasticidal : EN 13624 (dirtiness conditions – 0.5% – 5 min – 20°C), EN 14562 (dirtiness conditions – 0.5% – 15 min – 20°C),

Virucidal : EN 14476 on PRV*, BVDV*, H1N1, (dirtiness conditions – 2% – 5 min – 20°C) Herpès, Vaccinia virus, Rotavirus (dirtiness conditions – 0.5% – 5 min – 20°C), Norovirus (dirtiness conditions – 2% – 15 min – 20°C)

* Swine Pseudorabies Virus (PRV) and Bovine Viral Diarrhea Virus (BVDV) are surrogate of HBV and HCV.

PACKAGING

5L can

Réf. LB920300

SECURITY

For professional use only.

For more information please refer to the MSDS.